

SMART SEAL™ LINER

SAFR / SADT / SBFR

SAFR rings begin 5 inches from the center of the distal end.

SBFR rings begin 3 inches from the center of the distal end.

SADT rings for:

- Sizes 20-28 begin 3 inches from the center of the distal end.
- Sizes 32-38 begin 5 inches from the center of the distal end.



DESCRIPTION



The ALPS® Smart Seal™ Liner features raised bands that grip the socket wall to form a secure interface. The low modulus bands stretch against the socket wall, while the smooth inner wall conforms easily to the residual limb. The Smart Seal™ Liner has a new seamless knitted construction with a modern look and is available in locking and cushion suspension. The locking version features a new distal construction to control distal distraction.

| PART # | SUSPENSION | L-CODE |
|--------|------------|--------|
| SAFR | CUSHION | L5679 |
| SBFR | CUSHION | L5679 |
| SADT | LOCKING | L5673 |

Recommended for all activity levels.




TOP FEATURES

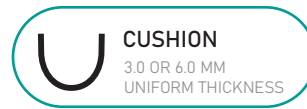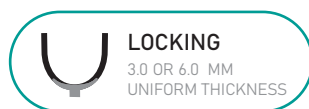


- Seamless knitted fabric
- Innovative single piece construction
- Offered with or without distal attachment

CROSS SECTION DIAGRAM


SADT 3.0 & 6.0MM UNIFORM

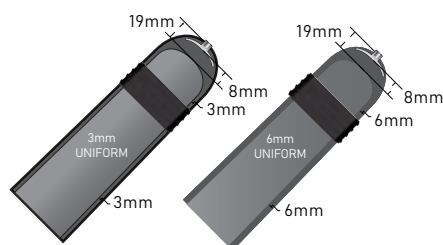

SAFR 3.0 & 6.0MM

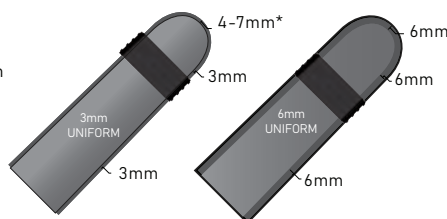


SBFR 3.0 & 6.0MM

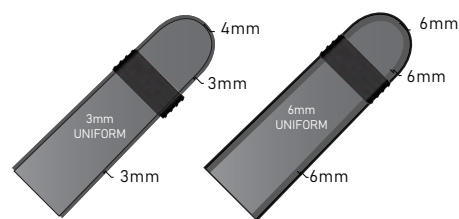

* SAFR sizes 26, 28, 32, and 35 have a distal thickness of 4mm, while size 38 has a distal thickness of 7mm.

HOW TO SIZE

Measurements taken
6cm from the distal end

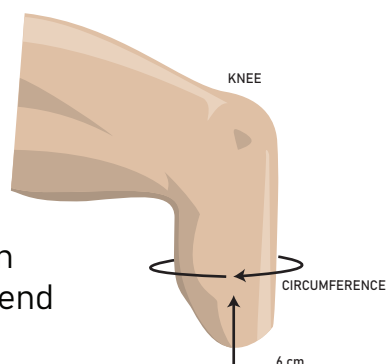

| MEASURED SIZE (CM) | SAFR - CUSHION 3.0MM or 6MM | SADT - LOCKING 3.0MM or 6MM UNIFORM |
|--------------------|--------------------------------|--|
| 20-23 cm | | SADT20 (3 or 6) |
| 24-25 cm | | SADT24 (3 or 6) |
| 26-27 cm | SAFR26 (3 or 6) | SADT26 (3 or 6) |
| 28-31 cm | SAFR28 (3 or 6) | SADT28 (3 or 6) |
| 32-34 cm | SAFR32 (3 or 6) | SADT32 (3 or 6) |
| 35-37 cm | SAFR35 (3 or 6) | |
| 38-43 cm | SAFR38 (3 or 6) | SADT38 (3 or 6) |

| MEASURED SIZE (CM) | SBFR - CUSHION 3.0MM or 6MM |
|--------------------|--------------------------------|
| 20-23 cm | SBFR20 (3 or 6) |
| 24-25 cm | SBFR24 (3 or 6) |
| 26-27 cm | SBFR26 (3 or 6) |
| 28-31 cm | SBFR28 (3 or 6) |

PRODUCTS OFTEN USED WITH SMART SEAL



VACUUM
INTEGRATED
PUMP (VIP)



SKIN RELIEVER (ENCP)

Use the Skin Reliever (ENCP) with the Smart Seal to compensate for volume fluctuations with continued use of same socket.

| MEASURED SIZE (CM) | ENCP TAPERED 3.0MM-1.5MM |
|--------------------|-----------------------------|
| 10-19 cm | ENCP16 |
| 20-27 cm | ENCP20 |
| 28-37 cm | ENCP28 |

*Expired patent owned by ALPS and/or one of its subsidiaries



T: 727.528.8566 | 800.574.5426 | F: 727.528.8862
www.easyliner.com | info@easyliner.com
2895 42nd Avenue North St. Petersburg, Florida 33714

