

SMART SEAL LINER

SAFR / SADT / SBFR

SAFR rings begin 5 inches from the center of the distal end.

SBFR rings begin 3 inches from the center of the distal end.

SADT rings for:

- Sizes 20-28 begin 3 inches from the center of the distal end.
- Sizes 32-38 begin 5 inches from the center of the distal end.



DESCRIPTION

The ALPS™ Smart Seal Liner features raised bands that grip the socket wall to form a secure interface. The low modulus bands stretch against the socket wall, while the smooth inner wall conforms easily to the residual limb. The Smart Seal Liner has a new seamless knitted construction with a modern look and is available in locking and cushion suspension. The locking version features a new distal construction to control distal distraction.



HIGH DENSITY
(HD) GEL



LOCKING
3.0 OR 6.0MM
THICKNESS



CUSHION
3.0 OR 6.0MM
THICKNESS

TOP FEATURES



Seamless knitted
fabric



Innovative single
piece construction



Offered with or without
distal attachment




#makinglivesbetter


MAKING LIVES BETTER

CROSS SECTION DIAGRAM


SADT 3.0 & 6.0MM UNIFORM

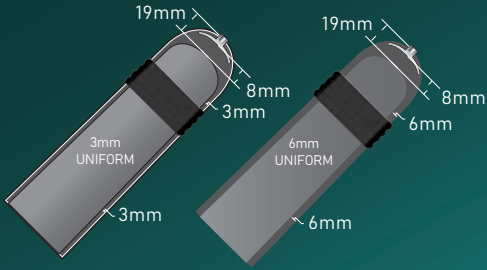

SAFR 3.0 & 6.0MM

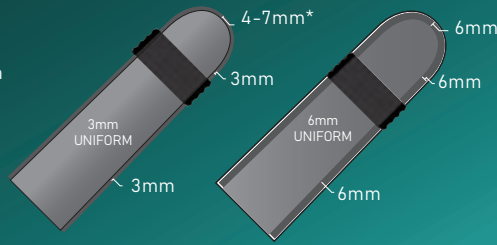


SBFR 3.0 & 6.0MM

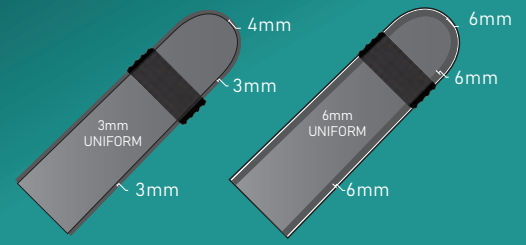

* SAFR sizes 26, 28, 32, and 35 have a distal thickness of 4mm, while size 38 has a distal thickness of 7mm.

HOW TO SIZE

Measurements taken 6cm from the distal end

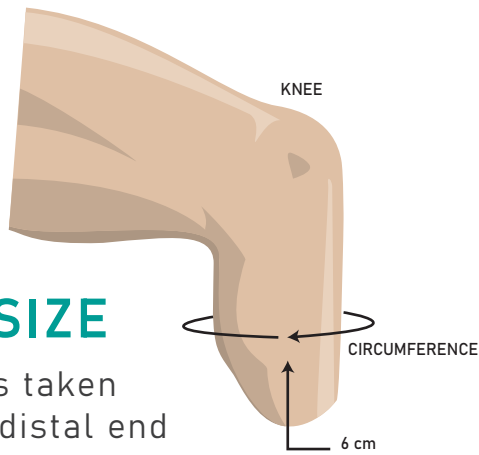

MEASURED SIZE (CM)	SAFR - CUSHION 3.0MM or 6MM	SADT - LOCKING 3.0MM or 6MM UNIFORM
--------------------	--------------------------------	--

20-23 cm		SADT 20 (3 or 6)
24-25 cm		SADT 24 (3 or 6)
26-27 cm	SAFR 26 (3 or 6)	SADT 26 (3 or 6)
28-31 cm	SAFR 28 (3 or 6)	SADT 28 (3 or 6)
32-34 cm	SAFR 32 (3 or 6)	SADT 32 (3 or 6)
35-37 cm	SAFR 35 (3 or 6)	
38-43 cm	SAFR 38 (3 or 6)	SADT 38 (3 or 6)

MEASURED SIZE (CM)	SBFR - CUSHION 3.0MM or 6MM
--------------------	--------------------------------

20-23 cm	SBFR 20 (3 or 6)
24-25 cm	SBFR 24 (3 or 6)
26-27 cm	SBFR 26 (3 or 6)
28-31 cm	SBFR 28 (3 or 6)

PRODUCTS OFTEN USED WITH SMART SEAL



VACUUM INTEGRATED PUMP (VIP)



SKIN RELIEVER (ENCP)

Use the Skin Reliever (ENCP) with the Smart Seal to compensate for volume fluctuations with continued use of same socket.

MEASURED SIZE (CM)	ENCP TAPERED 3.0MM-1.5MM
10-19 cm	ENCP 16
20-27 cm	ENCP 20
28-37 cm	ENCP 28



T: 727.528.8566 | 800.574.5426 | F: 727.528.8862

www.easyliner.com | info@easyliner.com

2895 42nd Avenue North St. Petersburg, Florida 33714

