



PDS-Padsil

Silicone Distal Pads



ALPS Padsil is made from the non-allergenic properties of silicone. ALPS Padsils provide more cushioning, softness and durability for your patient due to the unique compression set characteristics of silicone, these pads will not take a set even with prolonged hard use. Padsils can be used in practically every type of socket, even inside silicone suction sockets and liners. Padsils are machinable and can be shaped to fit with standard tools. ALPS Padsil (**PDS-xx**) is available in 5 sizes **S, M, L, XL, XXL**.

Sizing Instructions

Step 1:

Measure the circumference in cm at the beginning of the cylindrical section approximately 6 cm from the bottom of the distal end. This is the measured size.

Step 2:

Find the measured Size from Step 1 in the following chart. Read across to the find the recommended size.

Limb Circumference (cm)	Padsil
14-19 cm	S
20-23 cm	M
24-27 cm	L
28-31 cm	XL
32-35 cm	XXL

Cleaning the Padsil

Wash the padsil by hand with a mild soap; such as Alps Cleanser (PD 595). Allow them to dry before the next use. Be sure **not** to use any antibacterial or deodorant soap on the padsil for this may damage the product.

ALPS Satisfaction Guarantee

ALPS offers a no hassle return policy within **30 days** from date of purchase. If you are unsure of the proper size of Padsil to order, you can order two sizes from ALPS Customer Service. Once received, decide which size fits best. You can then call ALPS Customer Service for a Return Authorization Number and return the improper size for full credit of the product cost within 30 days from date of purchase (excludes freight charges).

Warranty

ALPS Padsils are warranted against manufacture's defects for **6 months** from the date of purchase. ALPS warrants only that its products will meet its specifications. **THERE IS NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR USE, NOR ANY OTHER EXPRESS OR IMPLIED WARRANTIES.** The user's exclusive remedy and ALPS South LLC sole liability is limited to the replacement of any product shown to be otherwise than as warranted. ALPS South LLC. will not be liable for incidental or consequential damages of any kind.

To obtain an Alps Return Authorization Number (RA#) call ALPS Customer service between the hours of 8 am and 5 pm EST and provide an ALPS representative with the following information:

- 1. Customer ID Number**
- 2. Invoice Number**
- 3. Date of Invoice**
- 4. Nature of the return**

The RA# must be displayed on the exterior of the returned item box or it will be refused at the dock.

Any products returned to ALPS after 30 days without manufacturing defects will be subject to a 25% restocking charge.



CE

EC	REP
----	-----

CEpartner4U
Esdoornlaan 13, 3951DB Maarn, The Netherlands
www.cepartner4u.com

900027 Rev J, CAN# 6651
December 2016
PDS

ALPS South LLC
2895 42nd Ave. N.
St. Petersburg, FL. 33714
USA
Tel: 1-727-528-8566
Fax: 1-727-528-8862
Info@easyliner.com
www.easyliner.com